

HOW TO ACHIEVE OUTSTANDING RESULTS IN MESOTHERAPY

1. Professional Training
2. Proficient Protocols
3. Precise Techniques
4. Effective Equipment & Supplies
5. Superior Quality Ingredients

TECHNIQUES FOR SUCCESS

Whether you chose to administer Mesotherapy manually or with a Mesotherapy gun the choice of injection technique is important.

Generally, according to the depth, working from the epidermis to the hypodermis, a distinction is made between one of these 3 techniques:

PAPULE: 1 to 2mm, with bevel upwards
NAPPAGE: 2 to 4mm, angle of 30° to 60°
POIINT BY POIINT: deep injections, 4 to 12mm

TIPS FOR PATIENT SATISFACTION

1. *Local Anesthetic* - Applied prior to treatment, not essential but may be helpful for sensitive patients.
2. *Ice* - Applied prior to treatment, ice will achieve the numbness of anesthetic and also help reduce swelling from PPC.
3. *Loose Clothing* - After treatment loose fitting clothing can increase patient comfort particularly after PPC.
4. *Massage* - Encouraged 72 hours after treatment and continued daily between treatments massage can be effective in reducing the instance of nodules.

TOOLS REQUIRED FOR EFFECTIVE MESOTHERAPY

Needles

4mm, 6mm and 12mm 27G needles
4mm, 6mm and 12mm 30G needles

Syringes

5cc to 20cc luer lock syringes
5cc to 20cc slip tip syringes

Multi Injectors

Circular Multi Injectors - 4mm - 7 needles
Circular Multi Injectors - 6mm - 7 needles

Linear Multi Injectors

- 4mm - 5 needles
Linear Multi Injectors
- 6mm - 5 needles

Injection Plates

Circular Injector Plates
- 7 connections
Linear Injector Plates
- 5 connections

Mesotherapy Gun



Mesotherapy Worldwide

C401 1-11 Hunter Street
Waterloo NSW 2022 Australia
P: +61 419 268 639
F: +61 2 9698 7952
info@mesotherapyworldwide.com

Affordable & Effective Mesotherapy Supplies

Mesotherapy Worldwide

www.mesotherapyworldwide.com



TIPS, TECHNIQUES
AND TECHNOLOGY

