

## How to operate your MesoMega gun.



31/03/2010 Copyright Mesotherapy Worldwide 2010 1

If you have just purchased your MesoMega gun CONGRATULATIONS! Following is some tips to ensure you get the most out of it and how to make the MesoMega your best assistant in mesotherapy treatment.....



31/03/2010 Copyright Mesotherapy Worldwide 2010 2


## MesoMega

Everything you need in one handy hard suitcase, ideal to store and carry your MesoMega. **One important thing: When you replace your MesoMega, please take care to reposition as it is designed for.** The top cursor has a spacing adapted on the right hand side.



31/03/2010 Copyright Mesotherapy Worldwide 2010 3


## MesoMega Elements

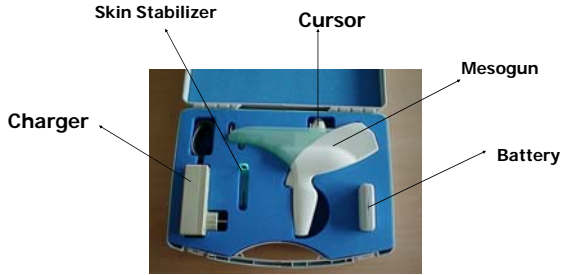

31/03/2010 Copyright Mesotherapy Worldwide 2010 4

## Major Functions of MesoMega

The control panel, will assist you to select and operate the various functions of injections, selection of syringes, but also to verify that the MesoMega gun is on or off, and what are the battery level.




31/03/2010 Copyright Mesotherapy Worldwide 2010 5

## MesoMega Basic

How to switch on and off your device:  
 First you need to remove the « green nose of MesoMega » by simply pressing at the top and clip it out.....Remember everything is very easy.....  
 When the « nose » is out you will see the spacing of the battery indicated with a plus and a minus sign, those signs are on the battery too. Insert the battery respecting the symbols and clip it down to secure.

**PLEASE REMEMBER THAT DIFFERENT BRANDS OF SYRINGES MAY NOT MATCH UP WITH THE NUMBERS ON THE GAUGE**

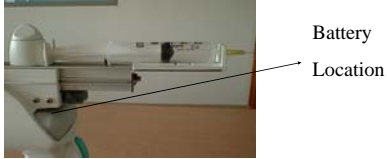

31/03/2010 Copyright Mesotherapy Worldwide 2010 6

## MesoMega – How to get started.

Now just replace the « nose », replacing the down part first following the guides, then move up the front to clip again the top forks of the « nose » to the rear of the gun. Then gently press the trigger once, you see on the control panel the battery level bars appearing. Now press « V » for couple of seconds and the control panel light up to show you some pre-selected functions.

Also one « green » light is on, under « C », you are now in continuous mode pre-select for using a 10ml syringe and inject at 5ml per minutes. The number 4 in the corner indicate you are going to use a 10ml syringe. See next chapter how to change the syringe model.

By pressing the arrows up or down you will see the injection ml per minute increasing or decreasing. It means you will inject faster or slower.



31/03/2010

Copyright Mesotherapy Worldwide 2010

7

## MesoMega Basic

The control panel is there to help you to select the mode of Mesotherapy treatment you want to work with,

To change mode press « S », each pressure on the key you will see the green light moving from continuous, dosimetric, nappage and mesoperfusion.

If now you press longer on « S » a Blue light will switch on... you are now entering on the syringe selection programmed



Continuous

Dosimetric

Mesoperfusion

Nappage

31/03/2010

Copyright Mesotherapy Worldwide 2010

8

## Select your syringe..



- You can adjust on MesoMega 5 types of syringes, by pressing the arrows up and down you will see the sizes going up and down. Please remember that in some programmed such as continuous injection the smallest one are not available... they will be empty after some seconds.
- **When you have selected your syringe press « V » to validate your choice.**
- Now you have return to the main menu you can see a figure on the right side of the display screen corresponding as follow
  - 1 for 1cc, 2 for 2cc, 3 for 5cc, 4 for 10cc, 5 for 20cc

31/03/2010

Copyright Mesotherapy Worldwide 2010

9

## MesoMega Set syringes

- You have now select a syringe and the program you want to operate with, so you just need to fix your syringe on your MesoMega, remember that you can choose from any brands, 3 pieces centered cone standard syringes.



31/03/2010

Copyright Mesotherapy Worldwide 2010

10

## Set your syringe...

- Please **have a look on the front part of the MesoMega** you can see some marking to indicate the positioning of the metal screw when changing syringe size. You will find an Allan Key in your gun kit that helps you to unscrew, and make loose the metal part of the syringe holding system. On the other side you have the cursor which is the second part of the syringe holding for the back of the syringe. This can be loose by just turning pulling up carefully the top of the cursor left or right. To lock again the cursor just pull back in position, the white part is fixed again.

The cursor can be locked and unlocked turning slowly the top only



Screw to be move with Allan Key inside the manual

Marking

31/03/2010

11

## Fix your syringe..

- Now take a close look at the syringe positioning.



**THIS IS HOW YOUR SYRINGE SHOULD BE FIXED**


31/03/2010

Copyright Mesotherapy Worldwide 2010

12

### A quick test to ensure syringe is secure...

- When you have set the syringe on the machine, if you want to be sure you have position fine the different part of the syringe body, and after you have locked the cursor again... just take the MesoMega and carefully put up side down, if the syringe is falling ... then you need to fix the syringe again.... If it doesn't move you have secured the syringe successfully.

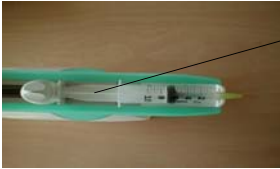

Please also check the position of the syringe piston the top of the cross should be position like that...

31/03/2010

Copyright Mesotherapy Worldwide 2010

13

### To operate the MesoMega

- Now your syringe is fixed, you have selected a menu, **continuous, dosimetric, nappage or mesoperfusion....**
- To start the treatment just press the trigger...To stop the treatment just release your finger from the trigger.



Trigger to operate the treatment, to stop just release your finger.

**NAPPAGE & CONTINUOUS DEMONSTRATION**

31/03/2010

Copyright Mesotherapy Worldwide 2010

14

### To order your MesoMega Mesotherapy Gun

includes Protocol Guide and Instructional DVD



phone: +61 2 9439 7781  
Fax: +61 8569 0915  
cell: +61 419 268 639

Email: [belinda@mesotherapyworldwide.com](mailto:belinda@mesotherapyworldwide.com)

31/03/2010

Copyright Mesotherapy Worldwide 2010

15